

# INSTRUCTIONS

## **HOW TO USE**

1. Position wheelchair (or chair) and client in front of sink with client's back to the sink.
2. Place a towel around client's neck and shoulders.
3. Place Tray strap over client's head with Tray behind head, placing neck opening around neck (FIG. 1).
4. Place client's forearm through strap and rest arm on abdomen to balance the Tray on shoulders (FIG. 2).
5. Make sure the end of the Tray is far enough over the sink for the water to drain off the Tray.
6. Wet hair and shampoo as usual. Rinse hair well, letting water flow down the Tray into the sink.
7. Towel dry hair.
8. Remove the Tray from client's shoulders.



**FIG. 1**



**FIG. 2**

## **CLEANING**

1. Hand wash Tray using warm water and mild soap, then rinse, towel dry, and store as needed.
2. **CAUTION:** If you have concerns about allergies, or how the recommended cleaning solution may affect your health, please consult with your physician or other healthcare professional before using so he/she can recommend alternatives, or, product can be disposed of after each use.

CE Technical Files kept at: 3 THE TROSSACHS, CLANCY STRAND, LIMERICK, V94 83H6, IRELAND  
© EZ-ACCESS, a division of Homecare Products, Inc. *All rights reserved.*

All text and images contained in this document are proprietary and may not be shared, modified, distributed, reproduced, or reused without the express written permission of EZ-ACCESS®.